

Copper Sulfate Solution

Copper Sulfate 10% w/v



WARNING: Fully read and understand all User Guides before using the Kelvi Pro[™] System. Failure to follow operating instructions could result in serious injury and voided warranty.

Refill Directions

1. Open Reservoir Cap on the top of Console by lifting up on the lever and pulling straight up to remove.

2. Pour distilled water into the reservoir until the water reaches the blue line or the Fill Reservoir screen reads "Full".

3. Add one (1) drop of Copper Sulfate Solution to reservoir each time water is refilled. This solution acts as an antimicrobial.

4. When the reservoir is full, replace Reservoir Cap and lower the lever to seal the valve.

5. The Kelvi[™] System will not operate if the Reservoir Cap is not properly installed.

This solution acts as an antimicrobial for your Kelvi[™] System and will keep the System clean and in proper working order. Failure to add Copper Sulfate Solution to the reservoir may void your warranty.

Warnings

- Do not ingest. If swallowed, seek medical attention or contact a Poison Control Center immediately.
- May cause skin irritation. If skin irritation occurs seek immediate medical attention. Wash contaminated clothing before reuse.
- Avoid release to the environment. Collect spillage.
 Toxic to aquatic life with long lasting effects. Dispose in compliance with local, state, & federal regulations.
- Do not use if solution changes color or becomes cloudy.
- Keep out of reach of children.
- Keep container closed when not in use.
- Do not store in direct sunlight.
- For use in the Kelvi[™] reservoir only.
- Failure to add Copper Sulfate Solution to the reservoir may void your warranty.

Copper Sulfate Solution



Hypothermia Devices Inc. 413 N. Oak St, Inglewood, CA 90302 Kelvi.com | (310) 492-5886

PNT-0008 Rev:00